

**Step 1**

Mark height  
Level rail

**Step 2**

Drill holes  
Use drill bit size 1/2"




**Step 3**

Prepare 1/4" toggle type  
wall anchors

**Step 4**


Insert 1/4" toggle type wall  
anchors in locations as  
marked



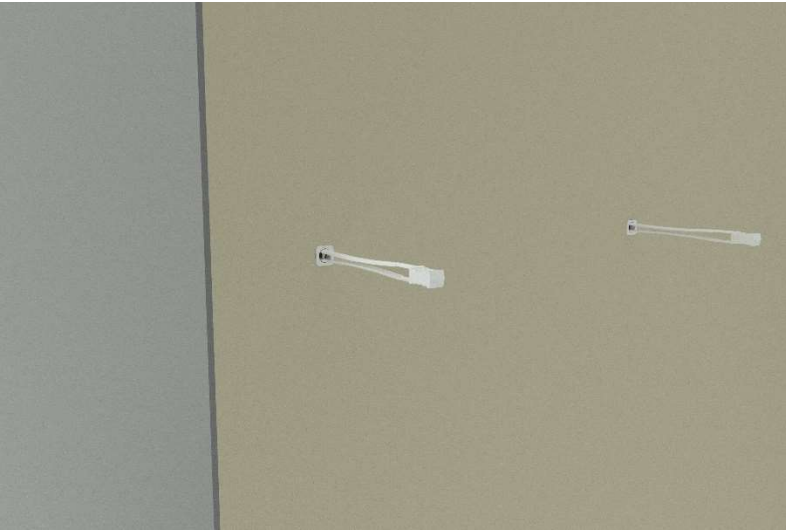


### Step 5

Align and tighten to the wall (see step 6)

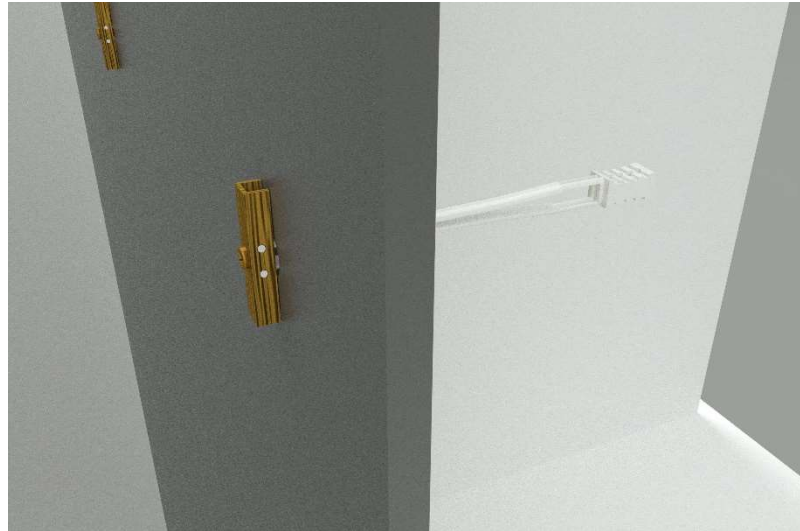


### Step 6

Image for reference

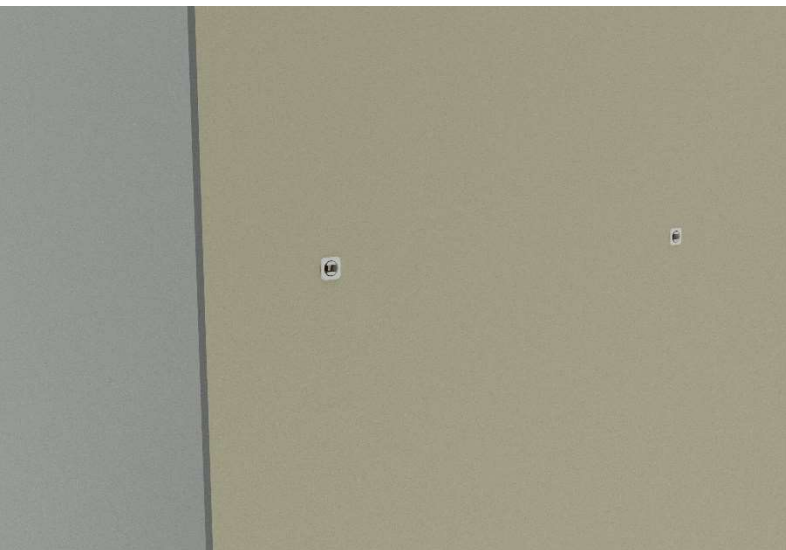


### Step 7

Bend until plastic cracks, flush to the wall

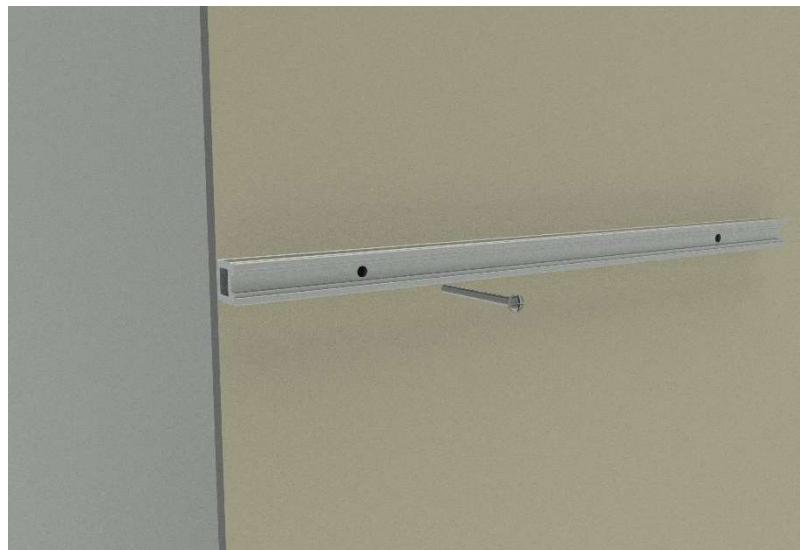


### Step 8

Re-align the rail.  
Insert and tighten ¼" screws

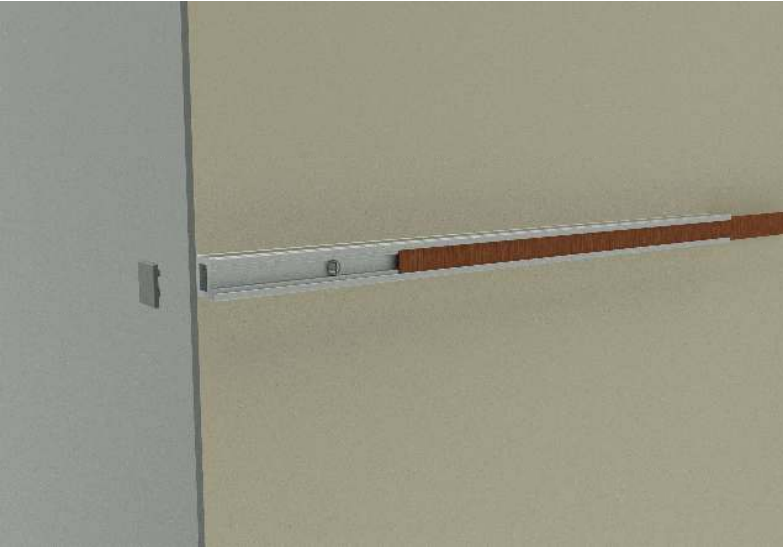


## Step 9

Insert provided inlay strip and secure end caps on both sides

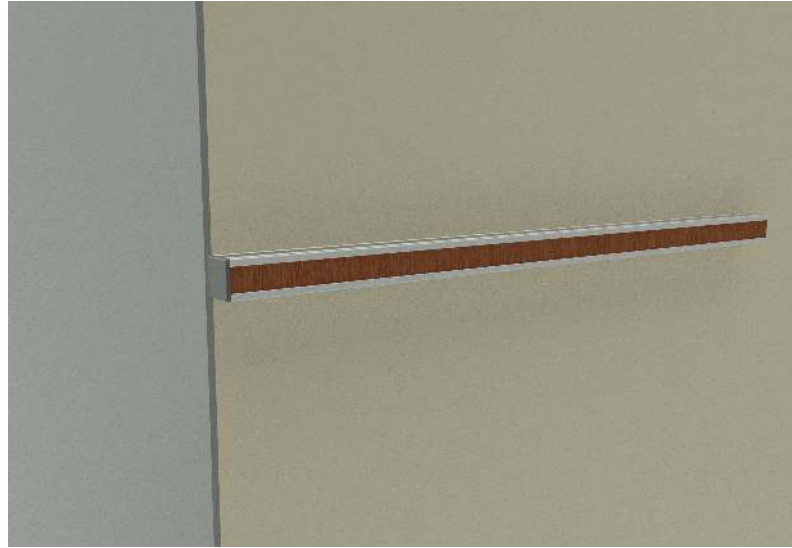

## Step 10

Finished product

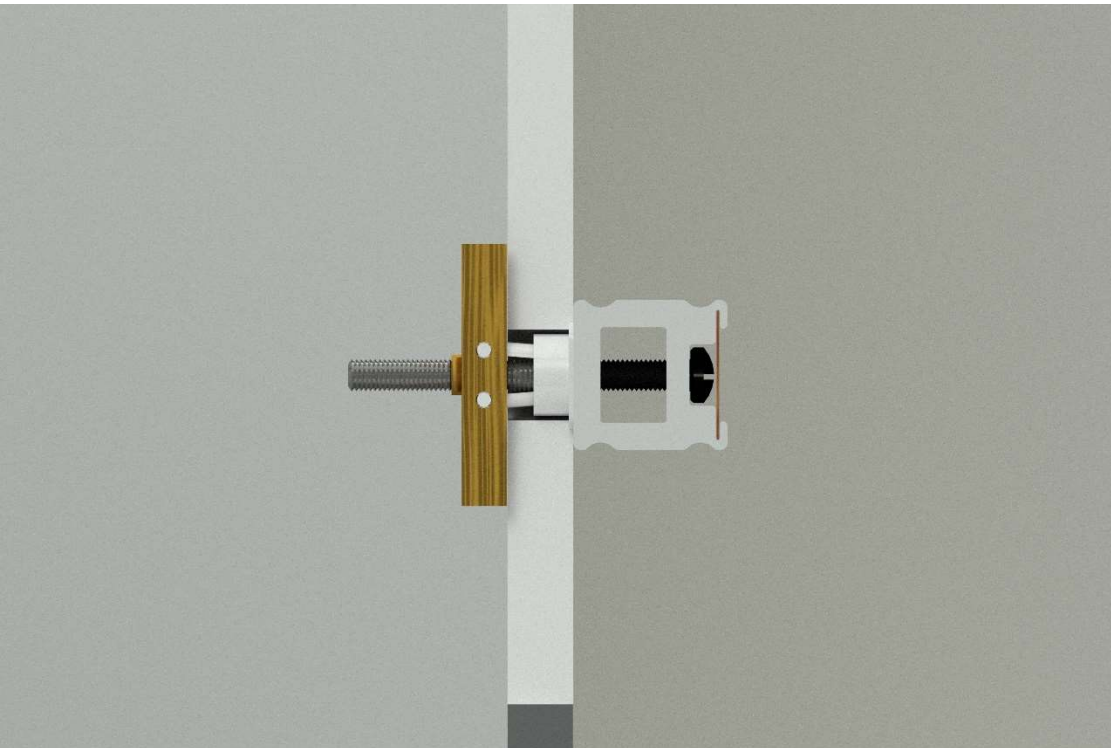

**CROSS  
SECTION  
OF THE  
FINISHED  
PRODUCT**

